








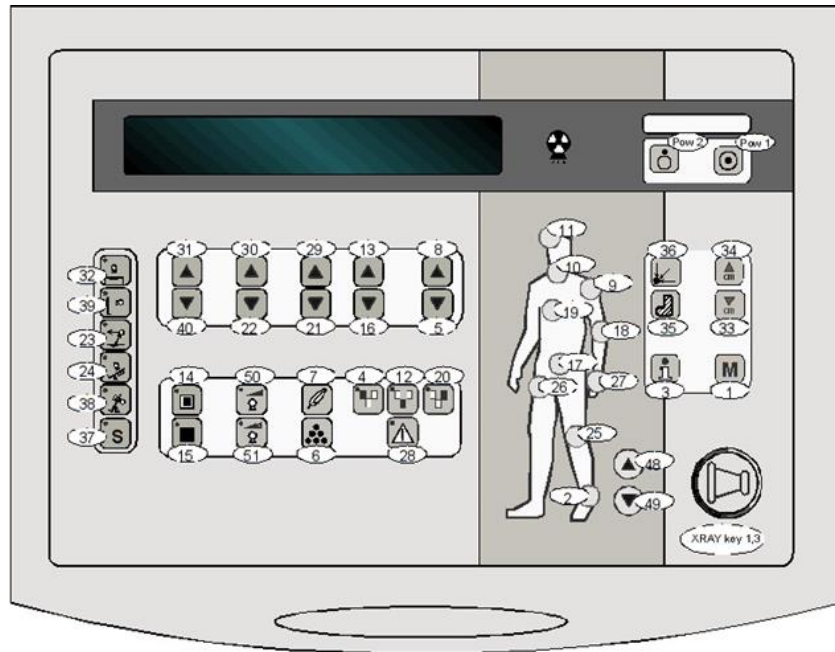

Call 800-333-5643 for service on this X-Ray system.



www.ChesapeakeMed.com






	Switching on generator	POW1 button
	Switching off generator	POW2 button



	1. Stage: Preparation ON 2. Stage: Exposure ON	Key No. X-ray 1, 3
--	---	--------------------



	Bucky table	Key No. 32
	Wall bracket	Key No. 39
	Tomography	Key No. 23
	Examinassions unit, conventionnel*	Key No. 24
	Examination unit, remote controlled*	Key No. 38
	Special unit*	Key No. 37



	Selection of small X-ray tube focal spot	Key No. 14
	Selection of large X-ray tube focal spot	Key No. 15

	100% tube power	Key No. 50
	≤ 100% tube power (default setting 70%)	Key No. 51


	The exposure technique can be switched over from 1-point technique to 2-point technique to 3-point technique by pressing the key several times.	Key No. 6
---	---	-----------


	To increase the tube voltage in steps of 1 kV (active in 1, 2 and 3-point technique)	Key No. 31
	To reduce the tube voltage in steps of 1 kV (active in 1, 2 and 3-point technique)	Key No. 40

	To increase the mAs product in stages (active in 2-point technique)	Key No. 30
	To reduce the mAs product in stages (active in 2-point technique)	Key No. 22

	To increase the tube current in stages (active in 3-point technique)	Key No. 30
	To reduce the tube current in stages (active in 3-point technique)	Key No. 22

	Error	Key No. 28
---	-------	------------

	Reset manually changed exposure parameters	Key No. 36
---	--	------------

	Save: Press "M" key Save for approx. 2 sec. to save adjusted techniques	Key No. 1
---	--	-----------